

**UNIVERSITY OF SOUTH ALABAMA
DEPARTMENT OF CARDIORESPIRATORY CARE
Clinical Competency Checklist
AMBU Bag Assembly**

Student's Name _____ Date _____ Attempt # _____

Rating Scale: **0 = inappropriate, no, incorrect, undone, absent**
 1 = needs additional study and practice
 2 = appropriate, yes, correct, done, present

<u>ITEM</u>	<u>RATING</u>
1. Disassembles AMBU bag.	_____
2. Cleans parts with attention to heavy soiled parts.	_____
3. Cleans manometer with calvicide only.	_____
4. Places parts of unassembled AMBU bag in Cidex for 20 minutes.	_____
5. Places parts of unassembled AMBU bag in rinse solution for 20 minutes.	_____
6. Places all unassembled parts of AMBU bag (excluding manometer) in dryer until fully dry.	_____
7. Reassembles AMBU bag.	_____
8. Pressure checks reassembled AMBU bag.	_____

Total = _____ out of 16

% = _____

70% proficiency required to pass

Instructor's Signature: _____

Instructor's Name: _____ Credential: _____
(Please print)

Student's Signature: _____