

**UNIVERSITY OF SOUTH ALABAMA
COLLEGE OF ALLIED HEALTH PROFESSIONS
DEPARTMENT OF CARDIORESPIRATORY CARE
Daily Blood Gas Analyzer Maintenance Checklist**

Name: _____

Date _____

Rating Scale: 0 = inappropriate, incorrect, or omitted
 1 = needs additional study and practice
 2 = appropriate and correct
 N/A = not applicable

<u>Item</u>	<u>Rating</u>
1. Runs protein remover if necessary.	_____
2. Runs cleaning solution if necessary.	_____
3. Checks and/or refills humidifier(s) as necessary.	_____
4. Checks fluid pack (which includes potassium chloride solution, calibration solutions, rinse solutions, and water receptacle) and replaces if necessary.	_____
5. Checks tank pressure, replaces if necessary.	_____
6. Cleans door/sample port.	_____
7. Records barometric pressure.	_____
8. Runs required control(s), takes any necessary corrective action records results.	_____
9. Records/notes all necessary data.	_____

80% proficiency required to pass Score = $\frac{x}{18} \times 100 =$ _____% _____ pass _____ fail

Instructor's Signature: _____

Instructor's Name: _____ Credential: _____
(Please print)

Student's Signature: _____

Comments _____
