

**UNIVERSITY OF SOUTH ALABAMA  
COLLEGE OF ALLIED HEALTH PROFESSIONS  
DEPARTMENT OF CARDIORESPIRATORY CARE**

**Volume Ventilator Assembly Checklist**

Name: \_\_\_\_\_ Date \_\_\_\_\_ Ventilator: \_\_\_\_\_

Rating Scale:           0 = inappropriate, incorrect, or omitted  
                          1 = needs additional study and practice  
                          2 = appropriate and correct  
                          N/A = not applicable

<u>Item</u>	<u>Rating</u>
1. Ventilator stripped of disposable equipment and decontaminated.	_____
2. Reusable equipment removed for decontamination.	_____
3. Mainstream bacteria filter replaced.	_____
4. Other filters replaced/cleaned as necessary.	_____
5. Hang new circuit from support arm.	_____
6. Install clean HME or humidifier module.	_____
7. Make circuit-humidifier-ventilator connections.	_____
8. Replace drain bottle and connect expiratory limb.	_____
9. Test ventilator per department procedure manual.	_____
10. Stock ventilator per department procedure manual.	_____

100% proficiency required to pass    Score =  $\frac{x}{20} \times 100 =$  \_\_\_\_\_%        \_\_\_\_\_ pass \_\_\_\_\_ fail

Instructor's Signature: \_\_\_\_\_

Instructor's Name: \_\_\_\_\_ Credential: \_\_\_\_\_  
(Please print)

Student's Signature: \_\_\_\_\_

Comments \_\_\_\_\_